# BACTIGRAS®

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**Üretici**  
Made in England by Smith & Nephew Medical Limited, Hull HU3 2BN  
\*Trade Marks of Smith & Nephew  
© Smith & Nephew 2003  
www.smith-nephew.com

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**Indications**  
BACTIGRAS is for use as a topical treatment only. It is indicated for a wide range of wounds where there is a risk of infection, or on already infected wounds in conjunction with systemic antibacterials. The range of wounds on which BACTIGRAS may be used include: • minor burns and scalds • lacerations, abrasions and other skin loss wounds • donor and recipient graft sites • leg ulcers.

**Instructions for use**  
1. Cleanse the wound using

Product Availability Code	Size	Availability
7456	5cm x 5cm	Carton of 50 dressings
7457	10cm x 10cm	Carton of 10 dressings
66003650	10cm x 10cm	Carton of 50 dressings
66003661	10cm x 40cm	Carton of 10 dressings
7461	15cm x 20cm	Carton of 10 dressings
66007505	15cm x 1m	Carton of 12 rolls

**Manufacturer**  
Made in England by Smith & Nephew Medical Limited, Hull HU3 2BN England  
\*Trade Marks of Smith & Nephew  
© Smith & Nephew 2003  
www.smith-nephew.com

# BACTIGRAS®

Chlorhexidine Acetate Tulle Gras Dressing

**STERILE R** Single use. Gamma irradiation sterilised. Do not use contents if pouch is opened or damaged. Store in a dry place (<25°C).

**Description**  
BACTIGRAS is a gauze of leno weave impregnated with White Soft Paraffin BP containing 0.5% Chlorhexidine Acetate BP. It is presented sterile either as individual dressings in peel-apart pouches or as rolls in pouches which can be opened by means of a tear-notch.

**Mode of action**  
BACTIGRAS acts as a barrier between the wound and a secondary dressing, allowing exudate to pass into the absorbent secondary dressing. The chlorhexidine acetate antiseptic is slowly released from the dressing, giving long lasting action against Gram positive and Gram negative bacteria.

**Indications**  
BACTIGRAS is for use as a topical treatment only. It is indicated for a wide range of wounds where there is a risk of infection, or on already infected wounds in conjunction with systemic antibacterials. The range of wounds on which BACTIGRAS may be used include: • minor burns and scalds • lacerations, abrasions and other skin loss wounds • donor and recipient graft sites • leg ulcers.

**Instructions for use**  
1. Cleanse the wound using standard techniques.  
2. Select the correct size dressing and peel open the pouch (or tear end from roll pack).  
3. Using gloves and forceps, remove protector papers (protector paper in case of roll pack).  
4. Lay over wound in a single layer in direct contact with the wound surface.  
5. Apply a secondary absorbent dressing if required and secure in place with a dressing retention bandage or dressing fixation sheet.  
6. Removal. BACTIGRAS may be left in place when the absorbent secondary dressing is changed. The frequency of changing will depend on the clinical circumstances and this can vary from more than once a day to twice per week.  
7. If the dressing sticks soak it off gently with sterile saline solution.

**Precautions for use**  
BACTIGRAS should not be used on more than 10% of the body area. It should not be used in known cases of sensitivity or allergy to chlorhexidine. Dermatitis, contact sensitivity or photosensitivity may rarely occur. Anaphylactic reaction to chlorhexidine has been reported, but is rare. Avoid contact with eyes, the middle ear, meninges and brain. BACTIGRAS is incompatible with soaps, other anionic materials (eg some skin cleansers), as well as potassium iodide. The use of BACTIGRAS with oxidising wound cleansing agents (eg Eusol, hydrogen peroxide and iodine solutions) should be avoided

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Made in England by Smith & Nephew Medical Limited, Hull HU3 2BN, England. \*Trade Marks of Smith & Nephew ©Smith & Nephew 2003 www.smith-nephew.com