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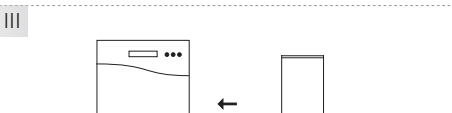
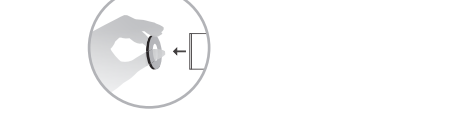
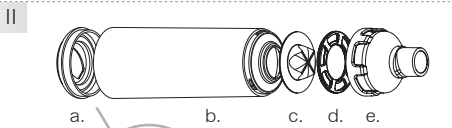
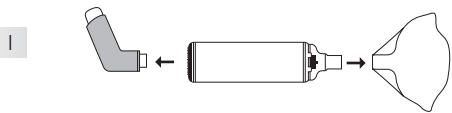
## CLINICAL REPROCESSING

### Cleaning / Disinfection

**I** Disconnect the metered dose inhaler (MDI). Optionally, when using a mask: remove the mask from the delivery chamber.

**II** Disassemble the RC-Chamber reusable completely into its 5 parts as shown in figure II. First remove the transparent plastic ring with the fixed integrated rubber adapter by inserting your thumb deep into the opening. Pull out the complete back piece with force. Afterwards disassemble the mouthpiece into the several parts (a.–e.).

**III** Place all the individual parts in the hospital dishwasher and select the cleaning program with 93° C (exposure time 5 minutes). Please observe the manufacturer's instructions. Use a pH-neutral detergent and observe the dosage recommendations by the manufacturer.

**IV** Lay the individual parts in their open position to air-dry. Ensure sufficient drying time. Do not rub them dry!

### Sterilization

**V** Sterilization is possible via the following procedures: Place all disassembled parts into a suitable sterilization package. Ensure that all the parts are dry. Sterilize the RC-Chamber reusable according to the steam sterilization process (DIN EN ISO 17665) using the following temperatures:

- max. 134 °C (reaction time at least 3 minutes)
- 121 °C (reaction time 15 minutes)

The recommended cooling time is 30 minutes.

Reassemble all the pieces according to figure II. Make sure that the mouthpiece is locked in place with a "click."

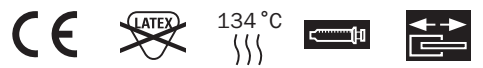
Disassembling the RC-Chamber reusable results in several pieces which can be swallowed by children. Therefore the individual pieces are always to be kept out of reach of children.

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Hersteller:  
Medical Developments International (MDI)  
4 Caribbean Drive,  
Scoresby, Victoria 3179,  
Australia

EU-Bevollmächtigter:  
EMERGO EUROPE  
Prinsessegracht 20  
NL 2514 AP, The Hague

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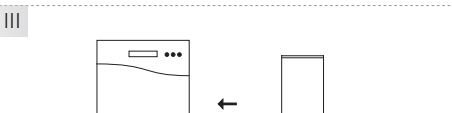
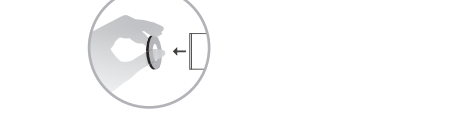
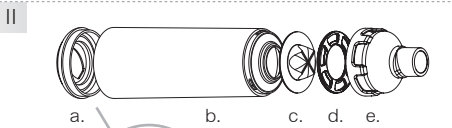
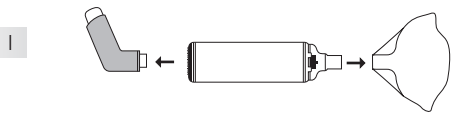
## QUICK START GUIDE Clinical Reprocessing

**I** Disconnect the metered dose inhaler (MDI). Optionally, when using a mask: remove the mask from the delivery chamber.

**II** Disassemble the RC-Chamber reusable completely into its 5 parts as shown in figure II. First remove the transparent plastic ring with the fixed integrated rubber adapter by inserting your thumb deep into the opening. Pull out the complete back piece with force. Afterwards disassemble the mouthpiece into the several parts (a.–e.).

**III** Place all the individual parts in the hospital dishwasher and select the cleaning program with 93° C (exposure time 5 minutes). Please observe the manufacturer's instructions. Use a pH-neutral detergent and observe the dosage recommendations by the manufacturer.

**IV** Lay the individual parts in their open position to air-dry. Ensure sufficient drying time. Do not rub them dry!

**V** Sterilization is possible via the following procedures: Place all disassembled parts into a suitable sterilization package. Ensure that all the parts are dry. Sterilize the RC-Chamber reusable according to the steam sterilization process (DIN EN ISO 17665) using the following temperatures:

- max. 134 °C (reaction time at least 3 minutes)
  - 121 °C (reaction time 15 minutes)
- The recommended cooling time is 30 minutes.

Reassemble all the pieces according to figure II. Make sure that the mouthpiece is locked in place with a "click."


Disassembling the RC-Chamber reusable results in several pieces which can be swallowed by children. Therefore the individual pieces are always to be kept out of reach of children.

## DISPOSAL

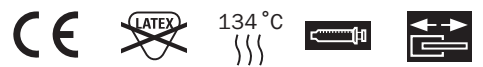Dispose the RC-Chamber reusable appropriately at the end of its product life.

## MATERIAL DURABILITY

RC-Chamber reusable can be reprocessed (sterilised) up to 40 times if no rinsing liquid is used. The RC-Chamber reusable as well as the mask should be replaced after 12 months.

In some instances, product graphics may vary from actual product models, sizes and/or colors listed below.

### For home care we recommend:

The RC-Chamber® models (with and without mask) optimize inhalation with MDIs, including emergency plan and guardian angel pendant.

## CEGLA MEDICAL TECHNOLOGY

Exclusive Distribution Germany, Austria and Switzerland:

R. Cegla GmbH & Co. KG  
Horresser Berg 1  
56410 Montabaur  
Germany  
Tel +49 2602 92 13-0  
Fax +49 2602 92 13-15  
www.cegla.de

Manufacturer:  
Medical Developments International (MDI)  
4 Caribbean Drive,  
Scoresby, Victoria 3179,  
Australia

EU-Authorized:  
EMERGO EUROPE  
Prinsessegracht 20  
NL 2514 AP, The Hague

## RC-CHAMBER REUSABLE

For hospital use

Please read the RC-Chamber reusable instructions for use carefully before first use. Please also follow the instructions of the nebulizer.

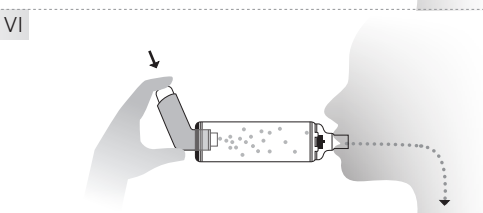
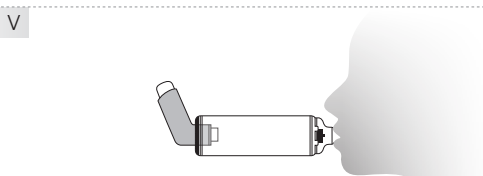
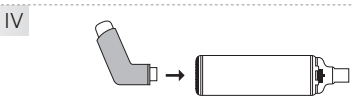
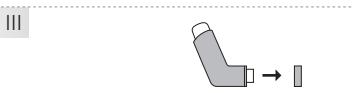
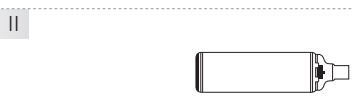
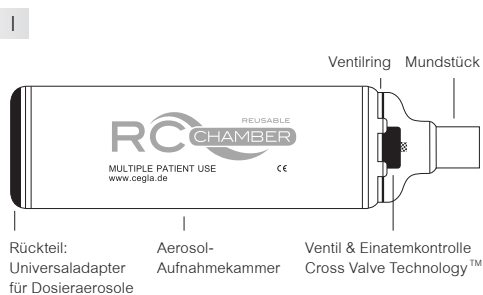
### The RC-Chamber reusable package contains:

- Mouthpiece
- Valve & inhalation control Cross Valve Technology™
- Spray entrance chamber
- Back piece: transparent plastic ring with permanently integrated rubber adapter
- The items 3071, 3081 and 3091 each include a mask







► A detailed clinical reprocessing instruction is available at hygiene@cegla.de

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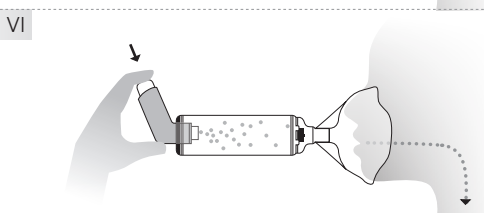
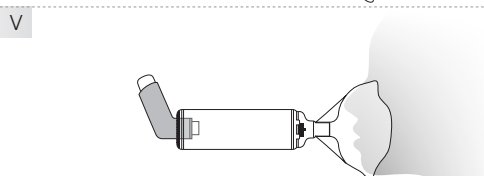
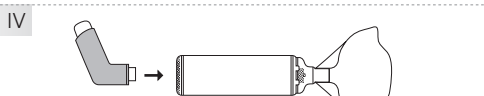
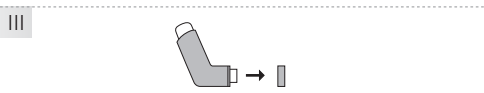
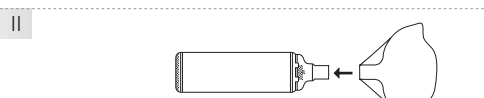
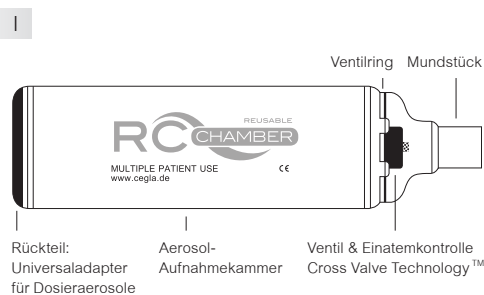
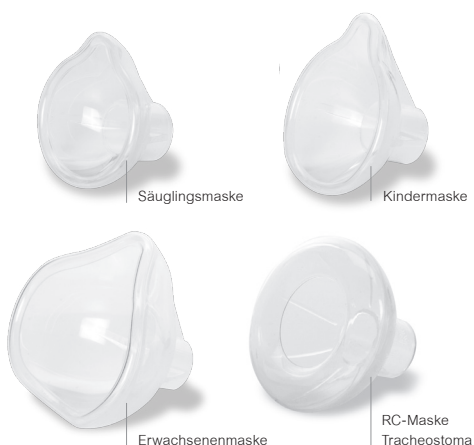
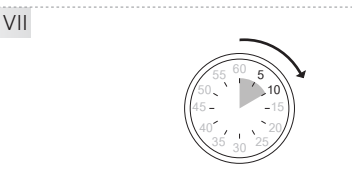
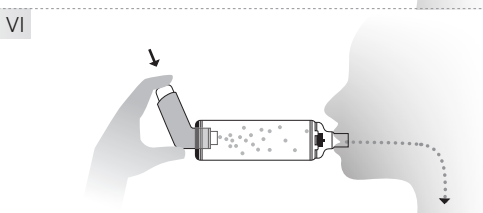
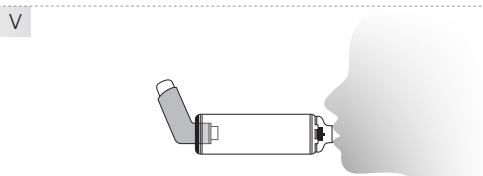
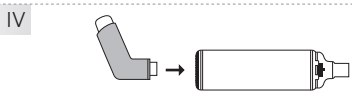
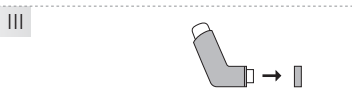
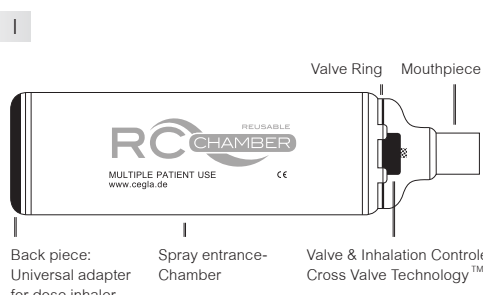
## QUICK START GUIDE with mask
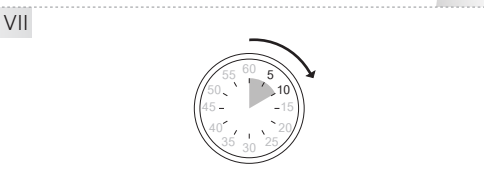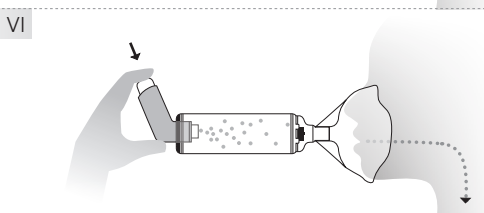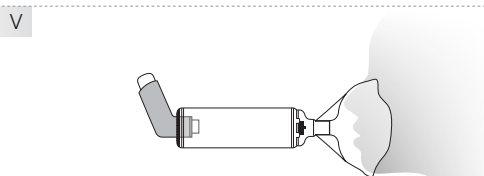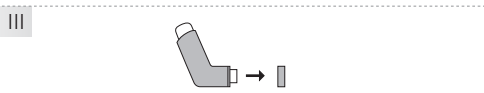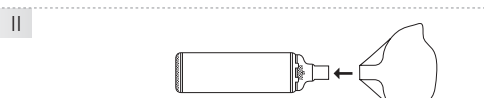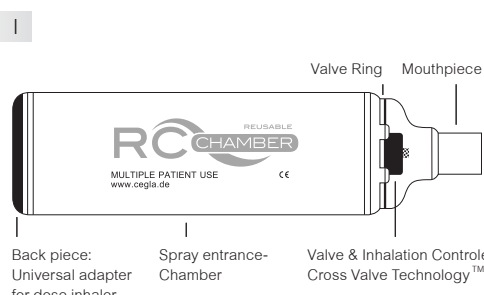
## INSTRUCTIONS FOR USE

I Continually check for foreign objects in the RC-Chamber reusable, for damage to any of the parts or if any parts are missing.

II For mask use: Place the mask on the RC-Chamber reusable mouthpiece.

III Remove the sealing cap from the mouthpiece of the dose inhaler and shake dose inhaler spray vigorously before each use.

IV Place the mouthpiece of the dose inhaler as shown in the figure in the provided rubber adapter of the RC-Chamber reusable.

V Place your lips tightly around the mouthpiece or place the mask airtight over the nose and mouth.

VI Push the cartridge of the dose inhaler down in order to dispense a spray into the chamber of the RC-Chamber reusable.

VII Breathe in and hold the breath for 5 to 10 seconds to optimize the effect. Children under 18 months should inhale 4 to 6 breaths, children and adults 2 to 4 breaths. Should several sprays be prescribed, then a one minute break should be taken before repeating steps V to VII!

## PURPOSE FOR USE

The RC-Chamber reusable is an inhalation device which was especially developed for MDI therapy with metered dose inhalers – so-called dose inhaler – and can be used for treatment of respiratory illnesses. The flexible rubber adapter is compatible with all common dose inhaler.

With the help of the RC-Chamber reusable the coordination between releasing the dose inhaler and inhalation of the medicine is omitted. In addition the large, not respirable particles are filtered out. Therefore irritation as well as a fungal attack in the mouth and throat is prevented.

## AVAILABLE PRODUCTS FOR THE CLINIC

### RC-Chamber reusable

- With mouthpiece: article-no. 3061EN
- With infant mask: article-no. 3071EN
- With child mask: article-no. 3081EN
- With mask for adults as well as older children: article-no. 3091EN

Infant mask: article-no. 3011EN

Child mask: article-no. 3021EN

Mask for adults and older children: article-no. 3041EN

RC-Tracheal mask: article-no. 3012EN